

The **Olympic Sterile-Driers** use bio-clean air for safer drying and packaging.

- Fast drying and large capacity
- Uses HEPA filtered air
- Flexible and convenient loading
- Protects the load while drying
- Provides short-term storage



**Model 44**



**Model 43**


# Recirculating heat and many other exclusive features

Olympic Sterile-Driers offer faster drying in bio-clean air, as recommended by infection control authorities. For safety and efficiency, they are the preferred driers following pasteurization or other methods of high-level disinfection.

By the use of HEPA-filtered air, Olympic Sterile-Driers eliminate the threat of airborne pathogens recontaminating wet, warm parts while they are being dried. All air entering the drying chamber first passes through a pre-filter and then a HEPA filter which is 99.97% efficient in removing all particles larger than 0.3 micron.

And Sterile-Driers can dry more parts faster. They can handle large loads with almost any combination of tubes and parts. Faster drying is achieved by the exclusive recirculating air system that efficiently recaptures heat instead of dissipating it.

These and many other useful features have made Olympic Sterile-Driers the first choice for bio-clean air drying of respiratory and anesthesia breathing tubes and parts.

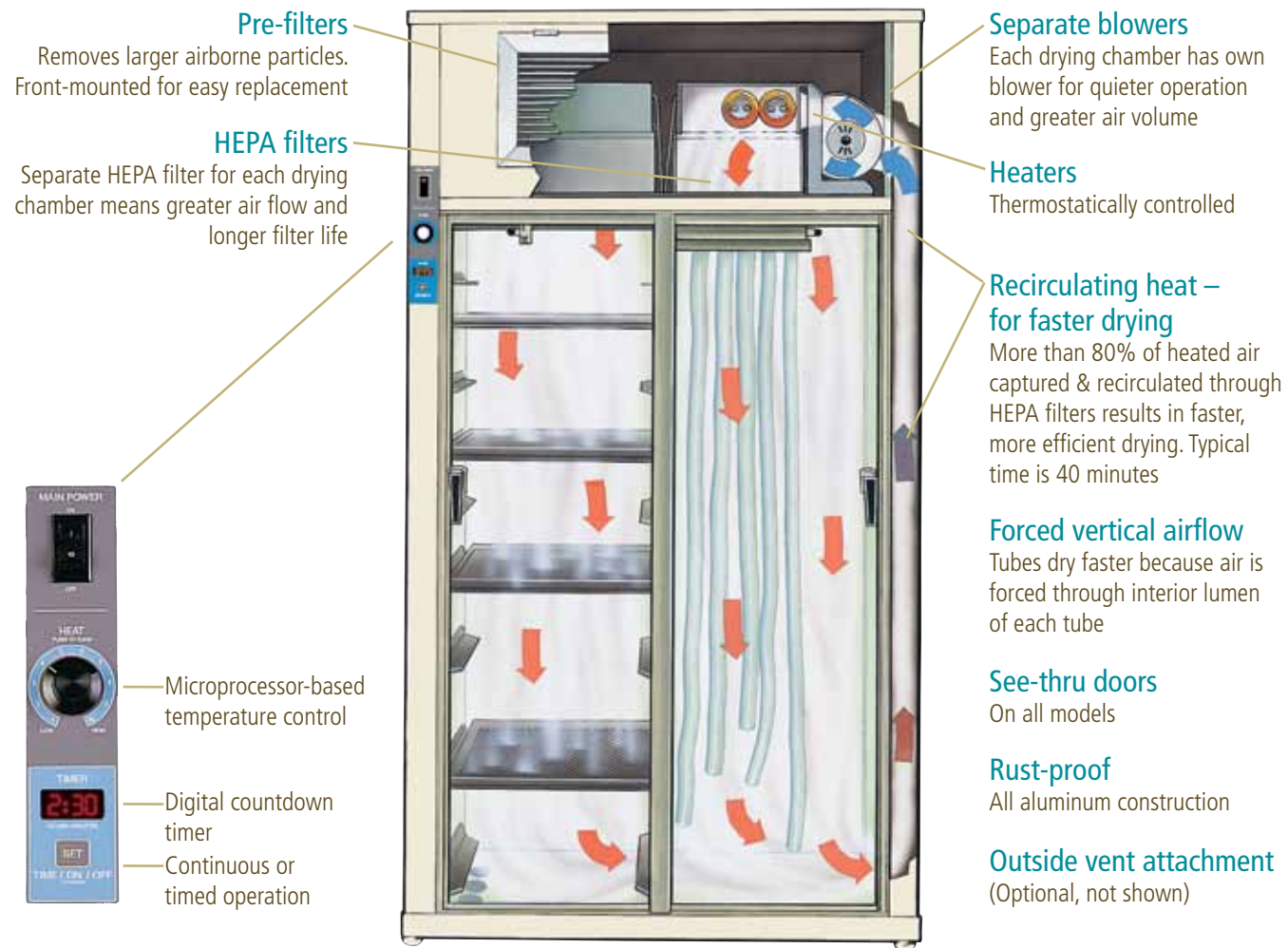

**Model 44**  
Large drying capacity can handle full load from an Olympic Pasteumatic. Model 44 shown with extra trays.



**Model 43**  
Fast drying of tubes and parts.

**HEPA filters**



Certified class 100 High Efficiency. Particulate air (HEPA) filters remove 99.97% of all particles larger than 0.3 microns that transport bacteria and viruses. Individually tested to assure entire HEPA filter system is leak-free.

**Removable trays**



Model 44 has nine large 18" x 18" (46 x 46 cm) parts trays standard. Model 43 has three 18" x 18" (46 x 46 cm) trays and six 18" x 9" (46 x 23 cm) half trays standard.

**Bag Drier**



Optional Bag Drier solves the problem of drying both the inside and outside of breathing bags after pasteurization. Slides into any Sterile-Drier model manufactured since 1984 and dries up to 16 bags at once.

**Slide-out tube holder**



Easy-loading tube-holders slide out at convenient height. Holds 35 tubes up to 5 ft. (1.5 m) long. Tube-holders can be replaced with more parts trays or Bag Drier.

# More good reasons for selecting Olympic Sterile-Driers

## Features and Specifications

### Safety features

Over-temperature thermal cut-offs on heaters and blower motors. Individually fused, 3-wire hospital-grade electrical connectors. Each Sterile-Drier individually tested for 1350-volt dielectric insulation

### Dimensions

Model 43: 26"W x 20.5"D x 86"H (66 x 52 x 218 cm)  
 Model 44: 46.5"W x 20.5"D x 86"H (118 x 52 x 218 cm)

### Electrical safety

CSA C/US certified to CSA C22.2 1010-1 and UL 61010A-1

### Electrical requirements

Available in the following configurations

Model 43	120V~, 60Hz, 8A
	100V~, 50Hz, 9.5A
	100V~, 60Hz, 9.5A
	220-230V~, 50Hz, 4.5A
Model 44	220-230V~, 60Hz, 4.5A
	120V~, 60Hz, 16A
	100V~, 50Hz, 18A
	100V~, 60Hz, 18A
	220-230V~, 50Hz, 9A
	220-230V~, 60Hz, 9A

### Department layout planning

Your representative can help you lay out an efficient decontamination area, including Sterile-Driers, and Olympic Pasteurmatrics

## Ordering Information

### Olympic Sterile-Driers

Model 43: With 3 full-size parts drying trays, 6 halfsize parts drying trays, and tube-holder for 35 tubes. 120V~, 60 Hz, 900 watts

**CAT. NO. 54343**

Model 44: With 9 full-size parts drying trays and tubeholder for 35 tubes. 120V~, 60 Hz, 1700 watts

**CAT. NO. 54344**

**Note:** Sterile-Drier models are available with optional rear doors and additional trays and tube holders

### Accessories

Bag Drier: For use with any Olympic Sterile-Drier manufactured since 1984

**CAT. NO. 54355**

For high-level disinfection, combine  
 pasteurization with bio-clean drying  
**Olympic Pasteurmatic™ 3000/3500**



Washes and pasteurizes in one completely automatic cycle, with minimal labor and handling.

## Information or Ordering

**Call 1-800-303-0306**

(toll-free in US/Canada)

**natus**

**Natus Medical Incorporated**  
 1501 Industrial Road  
 San Carlos, CA 94070 USA  
 1-800-303-0306  
 +1-650-802-0400  
[www.natus.com](http://www.natus.com)

© 2013 Natus Medical Incorporated. All Rights Reserved. All product names appearing on this document are trademarks or registered trademarks owned, licensed to, promoted or distributed by Natus Medical Incorporated, its subsidiaries or affiliates.